

Migration Disk Release 2 to Release 3

Overview

The Migration Disk provides an easy method of saving and restoring most of the settings and configuration files of an NP-1660 when upgrading from any release 2.x.x version of the operating software to release 3.x.x.


Description

The Migration Disk is an update disk which saves most of the settings and configuration of the printer onto the migration floppy disk when a release 2.x.x version of the operating software is detected, and restores them when release 3.x.x is detected. This procedure takes about 10 minutes.

Important: Once a migration disk is used on a printer, it should not be used on any other printer. Reusing a migration disk on a different printer will result in incorrect settings. Please contact Codonics Technical Support to receive a new migration disk for each printer being upgraded.

Saving Printer Settings

Locate the Migration Disk. The diskette will have the following label:

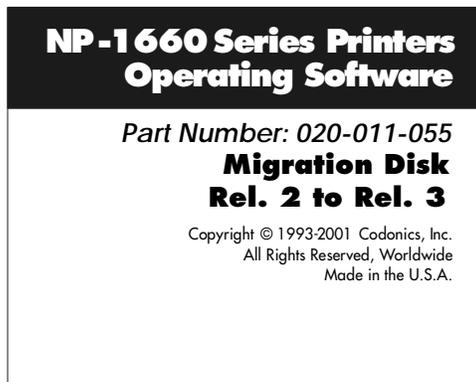

Follow the instructions below to perform the migration procedure. This procedure requires approximately 5 minutes prior to the Operating Software upgrade and another 5 minutes after the printer is upgraded.

1. Make sure the printer has media loaded and the Front Panel displays **READY**.

2. Position the printer so the floppy disk drive on the back panel is accessible.
3. Insert the Migration Disk into the floppy drive until the diskette "clicks" into the drive.
4. From the front panel, use the following menu sequence to save the migration data:

SETUP → UPDATE → YES

5. The printer will now save the settings of the printer to the floppy drive. When the Migration Disk has finished the printer will display **READY**.
6. Eject the Migration Disk from the floppy drive by depressing the eject button on the drive and mark the diskette with the date and printer that it was used on.
7. Print a Test Print page to have a hard copy of the settings to compare with after the update. To run a test print from the Front Panel, use the menu sequence:

READY → KEYS → T 1 _ _ _ _ _

8. Upgrade the printer with new operating software using the software installation procedure provided with the software. Return to step 9 on this document when the software upgrade is complete.

Restoring Printer Settings

9. To restore the settings and configuration files on an upgraded printer, put the Migration Disk into the floppy drive until the diskette "clicks" into the drive.
10. From the front panel menu, use the following menu sequence to restore the settings:

SETUP → UPDATE → YES

11. The printer will display **READY** on the front panel when the migration restore procedure is complete. Eject the Migration Disk from the floppy drive.
12. Print another Test Print from the front panel menu using the menu sequence:

SETUP → ADMIN → TEST PRINT → YES

13. Compare the before and after Test Prints and check that all the settings are restored or set them to the correct value from the front panel.

Technical Support

If the migration procedure is not successful or there are questions about using this procedure, please contact Codonics Technical Support department:

8:30AM - 5:30PM EST, Monday - Friday (Holidays Excluded)

Codonics, Inc.
17991 Englewood Drive
Middleburg Hts., OH 44130 USA

Phone: 440-243-1198
Fax: 440-243-1334
Toll Free: 800-444-1198
Email: support@codonics.com

Get it all with just one call
1-800-444-1198



CODONICS

We bring the future into focus

17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com