

Codonics Virtua Ink Cartridge Holder

Technical Brief

Ink Cartridge Holder Instructions

The following provides step by step instructions for use of an Ink Cartridge Holder to safely transport opened ink cartridges from location to location.

Preparation

- ◆ Identify how much ink remains in the cartridge by touching or clicking on the Virtua™ dashboard region located at the top of the User Interface. The systems status dialogue box will appear and provide the amount of ink remaining. Write this number down on a piece of paper.



- ◆ Remove ink cartridge from Virtua Ink Cartridge Carrier.

CAUTION! Hold cartridge by its side. Do not touch copper leads.



Sealing the Ink Cartridge

- ◆ Place the ink cartridge in the polybag and close tightly.



- ◆ Place the polybag in the bubble envelope. Fold the envelope flap top over but do not seal the bubble envelope so it can be reused.



- ◆ Place the bubble envelope in the ink cartridge holder.



- ◆ Close the ink cartridge holder tightly making sure that each of the four latches clicks.



Reusing the Ink Cartridge

- ◆ It is safe to travel with the ink cartridge holder in your carry-on baggage, in your checked baggage, or in your car.
- ◆ Do not store in temperatures in excess of 110°F or less than 34°F.
- ◆ When you reach your destination, remove the bubble envelope from the ink cartridge holder.
- ◆ Remove the ink cartridge from the bubble envelope. If ink leaks, wipe the cartridge with a lint free cloth.
- ◆ Install the ink cartridge back into the carrier as during normal setup.

Ordering Information

Catalog Number	Item	Qty
SP-00303	Codonics Virtua Ink Cartridge Holder	1
SP-00305	Codonics Virtua Ink Cartridge Polybag	5
SP-00319	Codonics Virtua Ink Cartridge Bubble Envelope	5

Technical Support

For questions or problems with the Codonics® Virtua Ink Cartridge Holder or any other Codonics product, please don't hesitate to contact Codonics Technical Support. 8:30AM - 5:30PM EST, Monday-Friday (U.S. holidays excluded):

Phone: 440-243-1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
1-800-444-1198



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2008 by Codonics, Inc. Printed in the U.S.A. Part No. 901-352-001 Rev. 02.