

# Codonics Safe Label System Drug Classification Templates— Custom Colors

### Summary

The Codonics Safe Label System® Administration Tool includes the ability to create drug classification templates based on existing templates but with custom colors. This Technical Brief describes how to use this feature.

### Creating a Drug Classification Template with Custom Colors

The Administration Tool includes a standard set of ISO- and ASTM-compliant drug classification templates. These are listed in the SLS Administration Tool User's Manual (Codonics Part Number 905-063-030).

You can create a new drug classification template that is based on one of the standard templates by customizing the background, line, and text colors.


**WARNING:** Creation of user-defined labels will deviate from ISO and ASTM standards governing user-applied labels. End users must take responsibility for confirming the legibility and usability of the resultant labels.

**NOTE:** Standard drug classification templates cannot be edited.

To create a drug classification template with custom colors:

1. With the Formulary drug list displayed, click a drug record Edit (  ) icon.

The Editing Formulary Drug(s) dialog box displays.

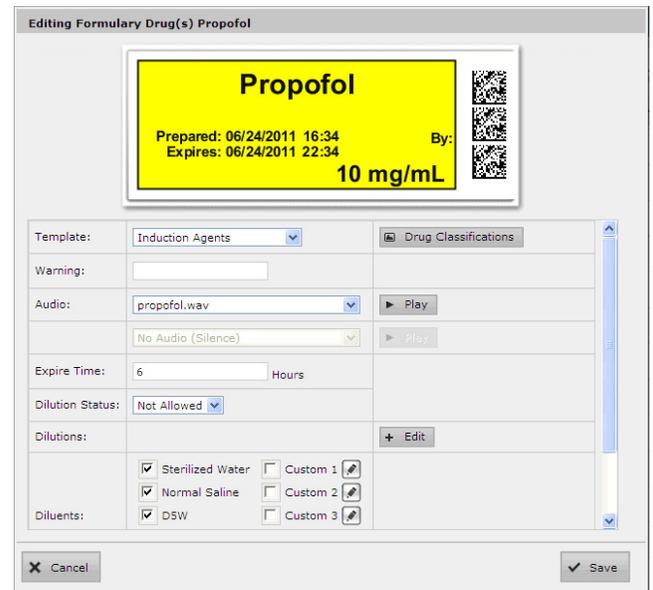


The screenshot shows the 'Editing Formulary Drug(s) Propofol' dialog box. At the top, there is a preview of a yellow drug label for 'Propofol'. The label includes the text: 'Propofol', 'Prepared: 06/24/2011 16:34', 'Expires: 06/24/2011 22:34', 'By:', and '10 mg/mL'. To the right of the label is a QR code. Below the preview is a configuration panel with the following fields and options:

- Template: Induction Agents (dropdown)
- Warning: (empty text field)
- Audio: propofol.wav (dropdown), with a Play button
- No Audio (Silence) (dropdown), with a Play button
- Expire Time: 6 (text input) Hours
- Dilution Status: Not Allowed (dropdown)
- Diluents: (checkboxes for Sterilized Water, Normal Saline, DSW, Custom 1, Custom 2, Custom 3)

At the bottom of the dialog are 'Cancel' and 'Save' buttons.


- Click the Drug Classifications button that is to the right of the Template list.

The Drug Classification Templates dialog box displays.

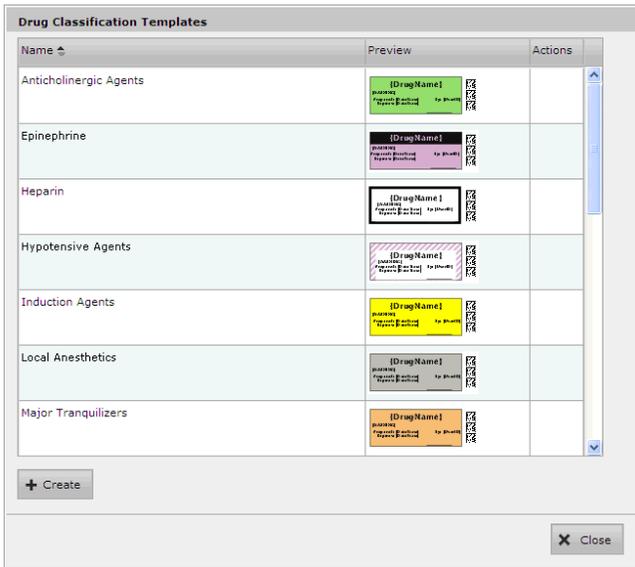

- Click the Create button.

The Create Drug Classification Template dialog box displays.

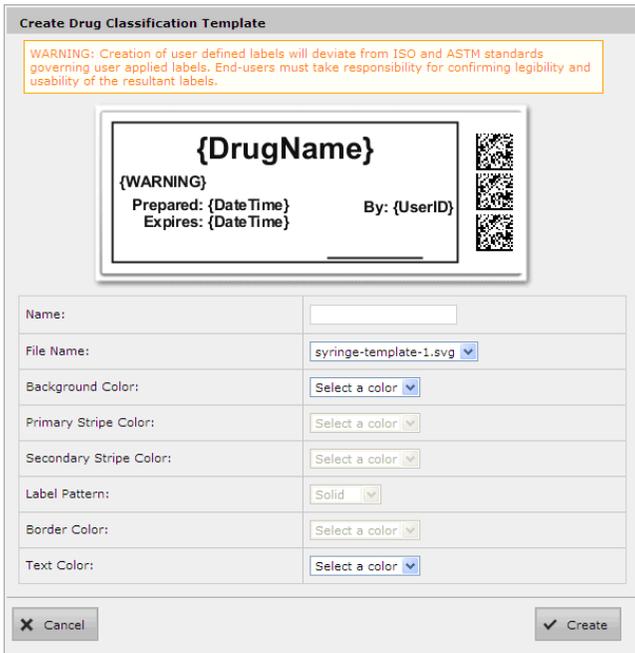

- In the Name field, enter a name for the new custom template.

- In the File Name list, select one of the standard templates on which to base the new template.


Standard templates are provided for specific drug categories, as listed in the following table.

Standard Template	Drug Categories
syringe-template-1	Induction Agents, Tranquilizers, Muscle Relaxants, Narcotics, Major Tranquilizers, Vasopressors, Local Anesthetics, Anticholinergic Agents
syringe-template-2	Antagonists
syringe-template-3	Succinylcholine, Epinephrine
syringe-template-4	Heparin
syringe-template-5	Combination drug (Miscellaneous White)
syringe-template-6	Protamine
syringe-template-7	Combination drug (Narcotic/Tranquilizer)

A preview of the selected label is displayed at the top of the dialog box. The lists of elements whose color you can select are enabled, depending on the standard template that you selected.


- Select the color for each of the enabled label elements.

The label preview updates with each color selection.

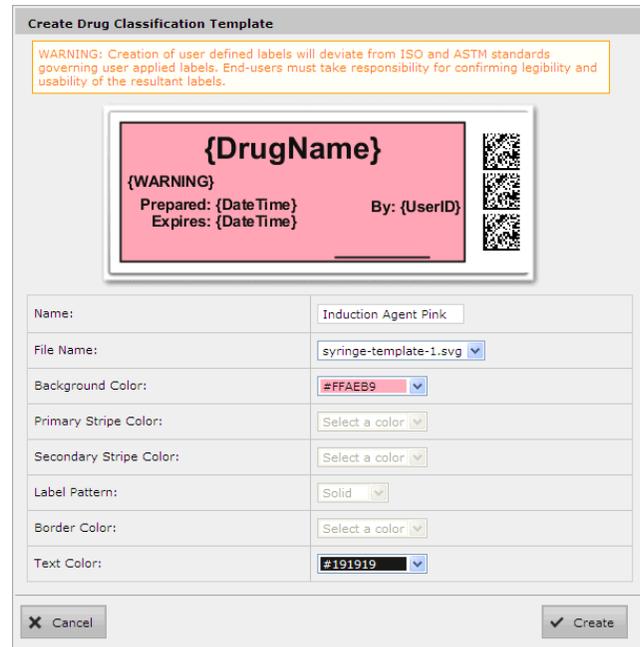


- Click the Create button to save the new template.

The new template is added to the list in the Drug Classification Templates dialog box.

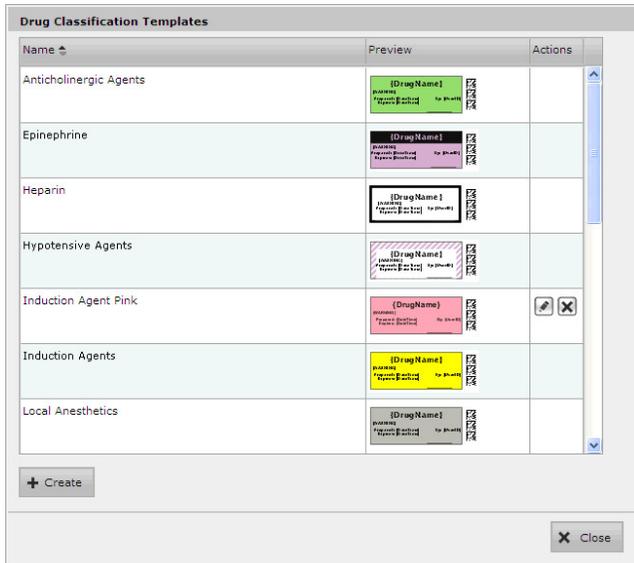

- Click the Close button to return to the Editing Formulary Drug(s) dialog box.

The new template is now included in the Template list.

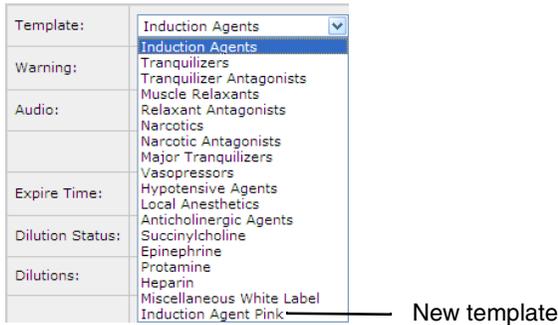

## Editing a Template with Custom Colors

You can edit a template with custom colors that you have already created.

**WARNING:** Editing the custom label will change the custom label for all drugs that use this label in the formulary. To identify all drugs that use this label template, sort the Formulary drug list by Template.

Template column

Master ID	Container ID	Name	Concentration	Tallman	Template	Actions
	0409904201	Bupivacaine-Epinephrine	0.25 % 1:200,000	Bupivacaine-EPINEPHrine	Miscellaneous W	

- On the Drug Classification Templates dialog box, click the Edit ( ) icon of the template that you want to edit. The Edit Drug Classification Template dialog box displays.

**Edit Drug Classification Template Induction Agent Pink**

**WARNING:** Creation of user defined labels will deviate from ISO and ASTM standards governing user applied labels. End-users must take responsibility for confirming legibility and usability of the resultant labels.

Editing the custom label will change the custom label for all drugs that use this label in the formulary. In order to identify all drugs that use this label template, sort in the SLS Formulary on Template.

{DrugName}

{WARNING}

Prepared: {DateTime}      By: {UserID}

Expires: {DateTime}

Name:	Induction Agent Pink
File Name:	syringe-template-1.svg
Background Color:	#FFAEB9
Primary Stripe Color:	Select a color
Secondary Stripe Color:	Select a color
Label Pattern:	Solid
Border Color:	Select a color
Text Color:	#191919

- After making your changes, click the Update button.
- The template colors are updated.


## Deleting a Template with Custom Colors

You can delete a template with custom colors that you have created.

**NOTE:** Standard drug classification templates cannot be deleted.

1. On the Drug Classification Template dialog box, click the Delete (  ) icon of the template that you want to delete.

You are prompted to confirm the deletion of the template.

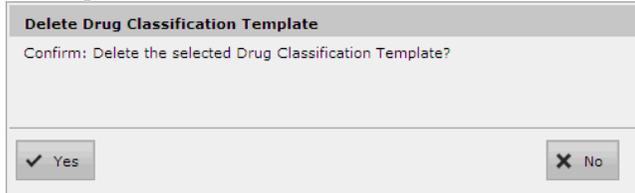

**Delete Drug Classification Template**  
Confirm: Delete the selected Drug Classification Template?

2. Click the Yes button to confirm the deletion, or the No button to cancel it.

## Technical Support

If problems occur when using a Safe Label System (SLS) or the Administration Tool, contact Codonics Technical Support.

Phone: +1 (440) 243-1198

Email: [support@codonics.com](mailto:support@codonics.com)

Website: [www.codonics.com](http://www.codonics.com)

---

*Get it all with just one call*  
**1.800.444.1198**



All registered and unregistered trademarks are the property of their respective owners.  
Specifications subject to change without notice.  
Patents: [www.codonics.com/ip/patents](http://www.codonics.com/ip/patents).

Copyright © 2010-2024 Codonics, Inc. Printed in the U.S.A. Part No. 901-248-003.02.

17991 Englewood Drive  
Cleveland, OH 44130 USA  
+1 (440) 243-1198  
+1 (440) 243-1334 Fax  
Email [info@codonics.com](mailto:info@codonics.com)  
[www.codonics.com](http://www.codonics.com)

Codonics Trading Co, Ltd.  
317 Xianxia Rd. Building B  
Unit 1412  
Changning Dist., Shanghai  
P.R. China, 200051  
86-21-62787701  
86-21-62787719 Fax

Codonics Healthcare Japan  
AQUACITY 9F,  
4-16-23, Shibaura  
Minato-ku, Tokyo, 108-0023  
JAPAN  
81-3-5730-2297  
81-3-5730-2295 Fax