

Codonics® SLS

Mounting Installation

Omnicell® Anesthesia Workstation

Overview

This document explains how to properly mount the Codonics Safe Label System® (SLS) Point of Care Station (PCS) onto an existing Omnicell Anesthesia Workstation (AWS) tabletop extension.

Parts List

The Codonics SLS PCS mounting accessory includes the parts listed in the following table. There are no tools required to mount the SLS PCS to the Omnicell AWS tabletop extension.

| Part | Part No. | Qty. |
|-------------------|-----------------|-------------|
| Mounting assembly | 572-192-001 | 1 |
| Thumbscrews | 550-457-XXX | 4 |

Location of the Mounting Assembly

The Codonics SLS PCS will be located on the right side of the AWS, as shown in the following figure.



Mounting the SLS PCS

1. Orient the mounting plate with the flat surface toward the base of the SLS PCS such that the feet of the device sit in the clearance holes on the plate.



2. Attach the mounting plate to the SLS PCS with the four supplied thumbscrews.



3. Set the SLS PCS and mounting plate on top of the existing shelf with the clips extending over the front and back.



4. Slide the front clip back so that it captures the underside of the shelf and tighten the front thumbscrew.



5. Slide the back clip forward so that it captures the underside of the shelf and tighten the back thumbscrew.



- Place the power brick inside the tabletop extension compartment, connect the device plug to the SLS PCS and plug in the 110V plug into an available outlet near the cart in the room.



Technical Support

If problems occur when using the mounting assembly with SLS PCS, contact Codonics Technical Support.

Phone: +1.440.243.1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
800.444.1198



All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Patents: www.codonics.com/ip/patent.

Copyright © 2025 Codonics, Inc. Printed in the U.S.A. Part No. 901-298-001.02.

17991 Englewood Drive
Cleveland, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Trading Co, Ltd.
317 Xianxia Rd. Building B
Unit 1412
Changning Dist., Shanghai
P.R. China, 200051
86-21-62787701
86-21-62787719 Fax

Codonics Healthcare Japan
AQUACITY 9F,
4-16-23, Shibaura
Minato-ku, Tokyo, 108-0023
JAPAN
81-3-5730-2297
81-3-5730-2295 Fax